


Cidox[®]+

High Level Sporicidal Cleaning and Disinfection Tablets

Preparing the Solution

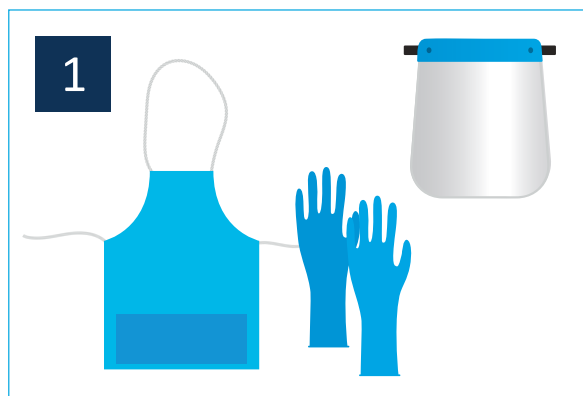

Disposable gloves, apron, eye and face protection are required at all times.



Remove any gross contamination before applying Cidox+.




Cidox+ contains detergent and does not require area pre cleaning.



Fill dilution bottle with lukewarm water (max 30°C). Add 1 Cidox 100+ to one litre bottle or one Cidox 500+ to five litre bottle. Allow tablet to fully dissolve to activate solution. Mark dilution bottle with time and date of makeup and user's initials.



Use solution according to your organisational policy and observe contact times. It is essential that a record is kept of when each solution is made.

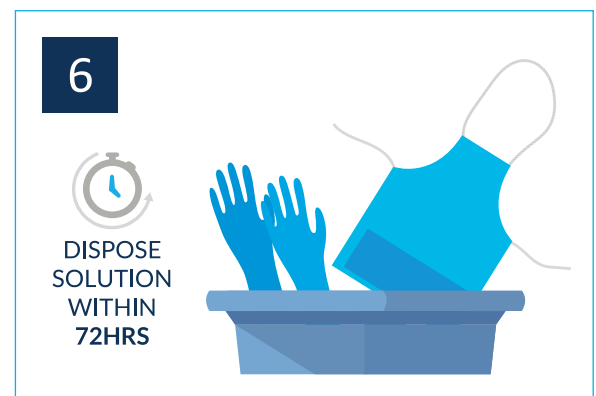

Cidox+ solution can be kept for up to 72 hours and any residual solution disposed of with cold running water. Rinse bottle and reuse. Dispose of PPE into correct waste stream and wash hands. Always refer to local policies for site specific instructions.

Cidox+ is compliant with all EN tests required by ISO 14885 for surface disinfection in the medical area.

A Technical Data and Material Safety Data Sheet are available on request.

! WARNINGS AND PRECAUTIONS

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE, AND REFER TO ORGANISATIONAL POLICY AND THE SAFETY DATA SHEET.



Avoid contact with skin and eyes.



Avoid continuous exposure to vapour.



Make up and use with adequate ventilation.



Use lukewarm water (max 30°C) to make Cidox+ solution.



Avoid prolonged contact with stainless steel, clothing or soft furnishings.



Use in accordance with your site policy and COSHH assessment. Store tablets and solutions in a secure, dry location, out of the reach of children.



Never remove tablet labels.



Always dispose of residual solution after 72 hours. Refer to local policies for site specific instructions.



Dispose of used materials in correct waste stream and wash hands thoroughly.